

SHAKIT™

B1C138 rev 00/13

THE DEVICE FOR EASY MIXING EVEN OF HIGH VISCOSITY RESINS AND BONE VOID FILLERS

CHARACTERISTICS:


- with SHAKIT the material can be mixed by **shaking** or using a spatula (based on the grammage and viscosity of the resin or bone filler)*
- mixing by shaking of high viscosity material is optimised by a **hammering ball** inside the device.



* SHAKIT mixes by shaking up to 20g of low/medium viscosity and up to 10g of high visc viscosity powder.

PREPARATION PHASES SHAKIT + XTRUDER

- 1 Pour the powder and liquid into the container and securely close the cap.
- 2 Forcefully shake the container for at least one minute until completely mixed.
- 3 Check that the scraper ring is connected to the end of the syringe.
- 4 Slowly push the syringe down to the bottom of Shakit to fill it completely.
- 5 Remove the syringe making sure that the scraper ring detaches from it.
- 6 Connect the syringe to the grip and turn it until you hear a click.
- 7 Assemble the extrusion screw to the grip.
- 8 Connect the application cannula.

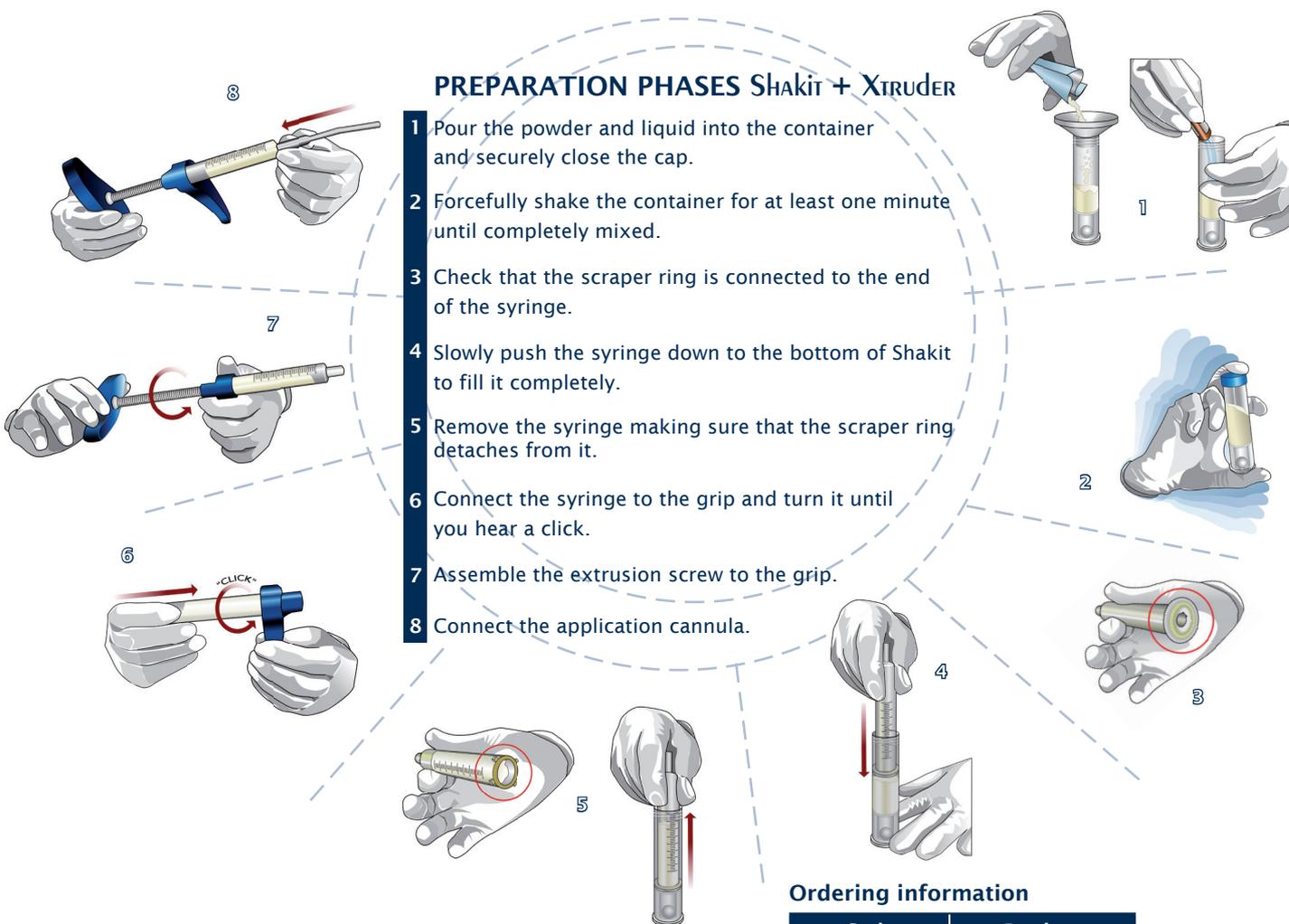

Ordering information

Code	Product
ASA0390	XTRUDER
ASA0380	SHAKIT

distributed by

